SK**ÚČEL URČENIA**

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Dátum poslednej revízie: 15.7.2025

EN**PURPOSE OF DESIGNATION**

Reagent designed for the rapid identification of carbapenemase producers.

SUMMARY AND EXPLANATION

Diagnostic kit CARBA is intended for fast identification of bacteria producing carbapenemases.

PRINCIPLE

By enzymatic hydrolysis of β -lactam ring, colouring of inoculated solution changes from negative red-pink to yellow, indicating positive test result of examined microorganism.

DIAGNOSTIC KIT CONTAINS

- CARBA „a“ - substrate in 5 pcs of microtubes eppendorf
- CARBA „b“ - solution min. 5 ml
- information leaflet

RECOMMENDED PROCEDURE

1. Add 5 - 6 drops of solution CARBA „b“ to microtube CARBA „a“. Put one full 10 μ l inoculation loop of bacteria mass to prepared microtube and thoroughly homogenise. Use fresh 24 h bacterial culture from suitable cultivation medium (for example blood agar).
2. Incubate in thermostat by 35 \pm 2 °C. Evaluate result of test after 0,5-2 hours.
3. If necessary, make control of result by simultaneous incubation without imipenem. Add 5 - 6 drops of CARBA „b“ to sterile tube and follow procedure according to point n. 1. and n. 2. If result colours after incubation are identical, consider test as negative.

For the need of own proof function, use following recommended bacterial strains:

Bacterial strain	CCM / ATCC	Colour	Interpretation
<i>Klebsiella pneumoniae</i>	- / BAA-1705	yellow	positive
<i>Escherichia coli</i>	4225 / 35218	red-pink	negative

CCM: Czech Collection of Microorganisms, Masaryk University Brno, Kamenice 5, 625 00 Brno, Czech Republic, tel. +420549491430, e-mail: ccm@sci.muni.cz

PERFORMANCE CHARACTERISTICS

Kit was assessed at NRC ATB ÚVZ SR on 88 bacterial strains with these results:

Specificity 100 % (95%CI: 92,38%-100%) and sensitivity 97,65% (95% CI: 87,10%-99,59%).

WARNINGS AND SECURITY PRECAUTIONS

- **For in vitro diagnostics use.**
- **For professional use only.**
- **Any serious adverse event occurring in connection with the device must be reported to the manufacturer and to the competent authority of the Member State (www.sukl.sk)**
- Follow the instructions exactly!
- All samples and inoculated products have to be considered as potentially infectious and obvious safety valid regulations must be respected while handling.
- Before use, check if the packaging is intact. Do not use damaged kit.
- Leftovers of expired or devaluated reagent liquidate according to regulations of your country. Packaging materials are recyclable.

STORAGE

Store CARBA at dark place at +2 to +8 °C. Expiration date is placed on the label of each package!

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