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Dátum poslednej revízie: 10. 3. 2014

**SUMMARY AND EXPLANATION**

INDOXYL is detection strip for based on detection of acetate esterase activity. Detection strip shows enzymatic activity by forming blue or blue-green colouration depending on metabolism of examining samples. Detection strip is used to presumptive identification of *Moraxella (B.) catarrhalis* or to differentiation of campylobacters.

**PRINCIPLE**

3-indoxylacetate as substrate of the reaction is enzymatically hydrolysed to form leuco form of indigo, which is oxidized by oxygen to blue or blue-green coloured chromogenic form of indigo indicating positive reaction of the test.

**DIAGNOSTIC KIT CONTAINS:**

- 50 test strips for 50 determinations
- information leaflet

**WARNINGS AND SAFETY PRECAUTIONS**

- **For *in vitro* diagnostics use.**
- **For professional use only.**
- Follow the instructions exactly!
- Used strips should be considered as potentially infectious and obvious valid safety regulations must be respected when handling.
- Observe common safety measures according to the regulations of your country.
- Before use, check if the packaging is intact. Do not use damaged kit.

**STORAGE**

At dark place at +2 to +8°C. Expiration date is placed on the label of package.

**REQUIRED MATERIALS**

Inoculation loop

**RECOMMENDED PROCEDURE:**

1. Take 2-3 colonies from pure 24 hours bacterial culture by an inoculation loop and spread them on the reaction zone of the detection strip. In case, zone can be moisten by small amount of water.
2. Let the strip incubate for 1-5 minutes by laboratory temperature on suitable plate, then evaluate the test.

**INTERPRETATION OF REACTION**

**Blue or blue-green colour indicates positive result of the INDOXYL test.**

**For the need of own proof function, use following recommend bacterial strains:**

Bacterial strain	CCM / ATCC	INDOXYL	Colour
<i>Moraxella catarrhalis</i>	4391 / -	+	blue / blue-green

CCM: Czech Collection of Microorganisms, Masaryk University Brno, Kamenice 5, 625 00 Brno, Czech Republic, tel. +420549491430, e-mail: [ccm@sci.muni.cz](mailto:ccm@sci.muni.cz)

*Tip: Use negative control - follow steps in recommended procedure, but do not use bacterial culture.*

*Use the content of package to 3 months from data of opening or to the end of expiration - if earlier.*

*In case when identifying campylobacters extend time of incubation to 25-30 minutes.*

Kit components doesn't contain hazardous substances.

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